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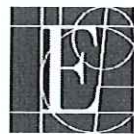
Edwards

© Copyright 2010, Edwards Lifesciences LLC
 All rights reserved.

Manufacturer 
Edwards Lifesciences LLC
 One Edwards Way
 Irvine, CA 92614-5686 USA
 Made in USA

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E Edwards Lifesciences

English

Bioprosthetic Valve and Annuloplasty Ring Accessories Care and Sterilization Instructions for Trays and Handles

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

1.0 Accessories

These instructions apply to all tray and handle models for bioprosthetic valves and annuloplasty rings. The instructions For Use supplied with Edwards Lifesciences bioprosthetic valves and annuloplasty rings provide information on the use of these accessories.

2.0 How Supplied

The accessories are packaged separately, provided non-sterile and must be cleaned and sterilized before each use. They cannot be sterilized in their original packaging.

3.0 Cleaning Instructions

All handles, trays, and lids must be cleaned separately and sterilized prior to each use. Accessories should be replaced on a routine basis. Contact your Edwards Lifesciences sales representative to obtain appropriate replacements.

CAUTION: Examine handles for signs of wear, such as delamination, cracking or crazing. Replace handle if any deterioration is observed.

WARNING: Fragments of the handles cannot be located by means of an external imaging device.

3.1 Instructions for Automated Cleaning:

Pre-rinse (as needed): Klenzyme enzymatic anionic detergent or equivalent.

Cleaning: Clean handles within tray and with lid detached prior to initial use and after each use with a nonionic detergent cleaning solution (example Instru-Klenz) in a mechanical washer (example STERIS AMSCO Reliance 444), minimum 2 minute wash phase.

3.2 Instructions for Manual Cleaning:

Pre-rinse (as needed): Per hospital procedure.

Cleaning: Place the handle, tray base, and tray lid in a cleaning solution bath, such as Cidezyme, an enzymatic detergent. For the time and temperature specified by its manufacturer. Ensure that the instruments are covered and do not touch each other. Clean the accessories thoroughly with a soft plastic brush for 5 minutes; remove any superficial impurities with the soft brush. Never use metal brushes or steel wool on the instruments. Always use fresh cleaning solution between cleanings. Afterwards, rinse each

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accessory thoroughly 5 times for 1 minute with sterile, deionized water.

Disinfection: Place the cleaned and inspected instruments in disinfection solution (example Cidex OPA) for the time and temperature specified by the manufacturer. Ensure that the instruments are covered and do not touch each other. Afterwards, rinse each instrument thoroughly 5 times for 1 minute with sterile, deionized water.

4.0 Sterilization Instructions

Threaded handles must be disassembled from any accessories before reesterilization.

CAUTION: Do not sterilize any of the accessories in their shipping containers. Handles must be removed from their plastic pouches prior to sterilization. Each institution should use procedures that include biological indicators to determine the effectiveness of the sterilization procedure.

CAUTION: Do not stack trays during sterilization.

All accessory models can be sterilized using the following recommended autoclave sterilization methods:

Gravity Displacement:

Wrapped:

Temperature 132 - 137 °C (270 - 279 °F)

Exposure Time 10-18 minutes

Unwrapped ("flash"):

Temperature 132 - 137 °C (270 - 279 °F)

Exposure Time 3-18 minutes

Prevacuum:

Temperature 132 - 137 °C (270 - 279 °F)

Exposure Time 3-18 minutes

Unwrapped ("flash"):

Temperature 132 - 137 °C (270 - 279 °F)

Exposure Time 3-18 minutes

The user is responsible for the qualification of any deviations from the recommended method of cleaning or sterilization.

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Edwards Lifesciences, the stylized E logo and Edwards are trademarks of Edwards Lifesciences Corporation; Edwards Lifesciences et le logo E stylisé sont des marques déposées auprès du Bureau des brevets et des marques commerciales des États-Unis (PTO).
Edwards Lifesciences, el logotipo estilizado de la E y Edwards son marcas registradas de Edwards Lifesciences Corporation. Edwards Lifesciences y el logotipo estilizado de la E están registrados en la Oficina de Patentes y Marcas de los EE. UU. (U.S. Patent and Trademark Office).
Edwards Lifesciences, Edwards en het logo met de gestileerde E zijn handelsmerken van Edwards Lifesciences Corporation; Edwards Lifesciences en het logo met de gestileerde E zijn gedeponeerd bij het U.S. Patent and Trademark Office.
Edwards Lifesciences och den stilerade E-Logotypen är registrerade hos U.S. Patent and Trademark Office (den amerikanska myndigheten för patent och varumärken).
Edwards Lifesciences, det stilerade E-Logo og Edwards är varumärket, der tilhører Edwards Lifesciences Corporation. Edwards Lifesciences og det stilerede E-Logo er registreret hos U.S. Patent and Trademark Office.
Edwards Lifesciences, o Estilizado do logótipo e Edwards são marcas comerciais da Edwards Lifesciences Corporation, Edwards Lifesciences e o Estilizado do logótipo estão registadas na Entidade de Marcas Registadas e Patentes dos EUA (U.S. Patent and Trademark Office).